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Druckversion: 2020-07

Content	Page
1. Information	30
2. Safety Instructions	30
2.1 Classification of the Safety Instructions	30
2.2 All Instructions for a Safe Handling of the Multi-Purpose Device	30
3. Application	31
4. Knee Joint System NEURO TRONIC	31
5. Adjusting the Orthosis	32
5.1 Checking the Orthosis' Basic Alignment	32
6. Putting the Multi-Purpose Device Into Operation	33
7. The Expert App	33
7.1 Starting Up the Expert App	34
8. Checking the Connection of Controller and Multi-Purpose Device	34
9. Checking the Connection between Controller and Expert App	35
10. Selecting a Mode	35
11. Signal Function for Training Purposes	36
12. Main Menu	37
12.1 Language	37
12.2 Pairing (Putting the Controller Into Operation)	37
12.3 Controller Update	38
12.4 Remote Control Update	39
12.5 Activate (Putting the Remote Control Into Operation)	39
12.5.1 For One Orthosis	40
12.5.2 For Two Orthoses	41
12.6 Cable Connection Test	42
12.6.1 Cable Connection Test Result Messages and Further Actions	42
12.7 Settings	42
12.7.1 Calibrate	43
12.7.2 Volume	43
12.7.3 Tone Selection	44
12.7.4 Signal Selection	44
12.7.5 Unlocking in Mid Stance	45
12.7.6 Locking in Terminal Swing	45
12.7.7 Rotation Safety	45
12.7.8 Select Other Leg	46

12.8	Battery Health	46
12.9	Step Counter	48
12.10	Device Info	49
13.	Indication of the Battery Status in the Multi-Purpose Device/App	49
14.	Replacing the Batteries of the Multi-Purpose Device	49
15.	Updating the Multi-Purpose Device/the Expert App	50
16.	Advice on Optimal Orthosis Functionality	50
16.1	Bluetooth Connection	50
16.2	Walking with the Orthosis in Auto Mode	50
16.3	Remote Control	50
16.4	Multi-Purpose Device	51
16.5	Controller	51
17.	Disposal	51
18.	CE Conformity	51
19.	Legal Information	51

1. Information

These instructions for use are addressed to orthotists or qualified/trained experts and do not contain any notes about dangers which are obvious to them.

2. Safety Instructions

2.1 Classification of the Safety Instructions

 DANGER	Important information about a possible dangerous situation which, if not avoided, leads to death or irreversible injuries.
 WARNING	Important information about a possible dangerous situation which, if not avoided, leads to reversible injuries that need medical treatment.
 CAUTION	Important information about a possible dangerous situation which, if not avoided, leads to light injuries that do not need medical treatment.
<i>NOTICE</i>	Important information about a possible situation which, if not avoided, leads to damage of the product.

All serious incidents according to Regulation (EU) 2017/745 which are related to the product have to be reported to the manufacturer and to the competent authority of the Member State in which the orthotist or qualified/trained expert and/or the patient is established.

2.2 All Instructions for a Safe Handling of the Multi-Purpose Device

WARNING

Risk of Falling Due to Improper Handling of the Multi-Purpose Device

Gather information about the correct handling of the multi-purpose device (e.g. avoiding contact with moisture and water). To do so, attend one of our production technique workshops, refer to the online tutorials on our website or contact Technical Support.

NOTICE

Damage to the Multi-Purpose Device's Batteries Due to Improper Handling

Use the multi-purpose device's batteries as described in these instructions for use.

Pay particular attention to:

- use the correct type of batteries;
- exchange them properly;
- carry out a battery quality test, if needed.

In case of persistent failure, contact Technical Support.

3. Application

The multi-purpose device is used to adjust the knee joint system **NEURO TRONIC**. This knee joint system is also available with Bluetooth® technology*. It must be handled by an orthotist or qualified/trained expert. With the multi-purpose device or the Expert app, you can adjust orthoses in which a **NEURO TRONIC** system knee joint is mounted.

In order to operate or adjust the orthosis via an app, the orthosis has to be equipped with a controller with Bluetooth.

4. Knee Joint System **NEURO TRONIC**

You have purchased one or more articles of the knee joint system. It consists of the following articles (fig. 1):

- 1 system knee joint
- 2 component set including controller
- 3 remote control for the patient including microprocessor-controlled quick charger or USB charging cable and User app
- 4 multi-purpose device for the orthotist or qualified/trained expert and Expert app

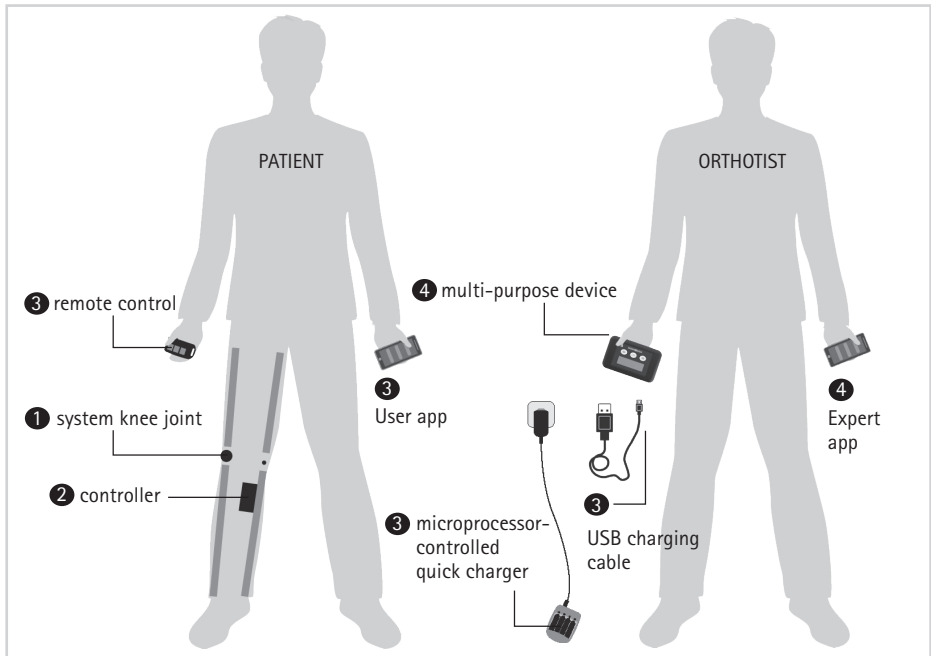
fig. 1

* The Bluetooth® word mark and logos are registered trademarks of Bluetooth SIG, Inc. and any use of such marks by FIOR & GENTZ is under license.

The system knee joint and the controller are mounted to the patient's orthosis. In order to put the orthosis into operation and adjust it, you need the multi-purpose device or the Expert app. The app has to be unlocked with a multi-purpose device with Bluetooth once. The patient needs the remote control to use the orthosis. As a complement, the User app can be used.

The multi-purpose device is delivered in a case. The case includes the following system components (fig. 2):

fig. 2

Item	Article Number	Description	Unit	Quantity
1	ET3840-T	multi-purpose device with Bluetooth including batteries	set	1
2	ET0710	USB charging cable	piece	1
w/o fig.	ET0830-2450	coin cells, CR2450	piece	2

5. Adjusting the Orthosis

Respect a proper alignment before putting the orthosis into operation. You can then make further adjustments to the orthosis using the multi-purpose device/app.

5.1 Checking the Orthosis' Basic Alignment

On the Workbench

Regardless of the plantar flexion, automatic system knee joints require a systematic adjustment of the dorsiflexion stop for an optimal function of the orthosis. The dorsiflexion stop determines the moment the system knee joint unlocks during mid stance. Furthermore, it affects the extension moment which is applied to the orthosis and the system knee joint, respectively. This is necessary to unlock the system knee joint.

Fix the foot piece of the orthosis firmly in the patient's shoe and put the orthosis on the workbench. The dorsiflexion stop of the system ankle joint must be adjusted in such a way that the line of gravity passes through the middle of the femoral shell and runs vertically downwards in front of the system ankle joint and between the ankle's pivot point and the rolling-off line.

Statically on the Patient

For checking the correct static alignment of the orthosis, the patient must wear the orthosis and stand upright with parallel feet. When viewed from the side, the line of gravity must run from the body's centre of gravity vertically downwards in front of the system ankle joint and between the ankle's pivot point and the rolling-off line. The course of the line of gravity at knee height results from the individual normal posture. Wearing the orthosis leads to deformation of soft tissue. This deformation causes the line of gravity to shift forward. Please consider this by readjusting the dorsiflexion stop, if necessary.

If the dorsiflexion stop is adjusted correctly, a lever between forefoot and lower leg is formed (activation of the forefoot lever) which brings the patient into a stable balance (they are able to balance themselves) and applies the necessary knee extension moment.

Dynamically on the Patient

For checking the correct dynamic alignment of the orthosis, the patient must wear the orthosis and walk a few steps with it. The dorsiflexion stop must be adjusted in such a way that a heel lift can clearly be seen in terminal stance. The consequence is a lever between forefoot and lower leg which brings the patient into a stable balance and applies the necessary knee extension moment. If the heel does not lift, you must reduce the system ankle joint's range of motion in dorsiflexion.

6. Putting the Multi-Purpose Device Into Operation

The multi-purpose device is delivered with batteries and can immediately be put into operation.

Briefly press the Set button to put the device into operation. You can now see the three modes AUTO, FREE and LOCK on the display (fig. 3). The available modes can not be selected until the multi-purpose device is connected to the controller (see paragraph 12.2).

fig. 3

When the batteries of the multi-purpose device are almost empty, a message will be displayed (fig. 4). In order to conserve the batteries, the multi-purpose device turns off when it is not used for longer than 10 minutes.

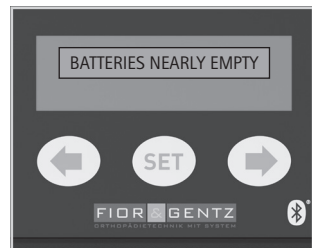
fig. 4

7. The Expert App

The app provides the same functions that you can use with the multi-purpose device.

If a controller with Bluetooth is mounted to the patient's orthosis, you can adjust the orthosis either with the multi-purpose device with Bluetooth or with the free app (fig. 5) via your smartphone/tablet. Minimum requirements are Bluetooth 4.0 and Android 5.0 or iOS 10.

fig. 5

7.1 Starting Up the Expert App

Download the app with your smartphone/tablet and connect the app to a multi-purpose device with Bluetooth once to unlock it. This way, we ensure that patients cannot access the Expert app and change the orthosis' settings.

In order to unlock the app with the multi-purpose device, proceed as follows:

- 1 Select the menu item **Activate App** in the multi-purpose device (fig. 6) by browsing with the left and right arrow buttons.
- 2 Confirm your selection by pressing the Set button.
- 3 Open the app with your smartphone/tablet and follow the app's instructions.

fig. 6

The number of apps that can be unlocked with one multi-purpose device is unlimited. This means that an unlimited number of smartphones/tablets with the Expert app can be unlocked with one multi-purpose device.

8. Checking the Connection of Controller and Multi-Purpose Device

The controller gives signals in order to inform you about the connection status of the multi-purpose device/app and the controller. The second LED at the controller indicates that the multi-purpose device/app and the controller communicate. If the mode selection is displayed on the multi-purpose device, the connection between the multi-purpose device and the controller is interrupted after 5 seconds. If you are in one of the other menu items of the multi-purpose device, the connection remains established and it is not possible to connect the app or the remote control to the controller.

Pressing the Key	Light Signal	Meaning
 <p>One of the three modes is selected.</p>	<p>colour: blue</p> <p>signal duration: ■</p> 	<p>The controller communicates with the multi-purpose device/app.</p>

If you have operated the orthosis with the app, you have to close the app in order to operate the orthosis with the multi-purpose device.

9. Checking the Connection between Controller and Expert App

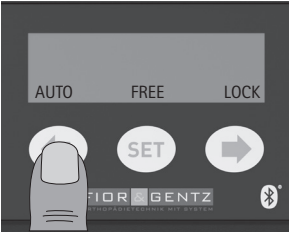
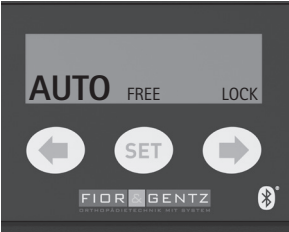
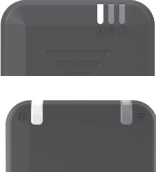
In order to adjust an orthosis with the app, Bluetooth must be permanently activated and the app must be opened in the foreground. Use the app menu and select the menu item **Pairing**. Follow the further instructions from the app. The controller cannot communicate with more than one Expert app at the same time. If there is an active connection to the app, the blue LED on the controller blinks permanently and the controller cannot be operated with the multi-purpose device. If you want to adjust the orthosis with the multi-purpose device or the Expert app on a different mobile device, you must first close the app currently connected to the controller.

10. Selecting a Mode

You can select the available modes Auto, Free and Lock with the multi-purpose device/app. The mode is set, when the respective mode is displayed larger in the multi-purpose device than the others.

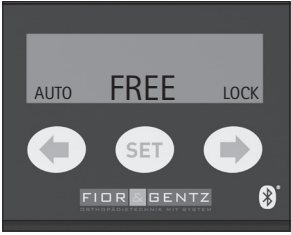
In order to change the mode with the multi-purpose device, proceed as follows:

- 1 Select the Auto mode by pressing the left arrow button. The  LED mode indicator at the controller blinks green.

Pressing the Key	Display after Pressing the Key	Light Signal		
		<p>colour: green</p> <table border="1"><tr><td>signal duration:</td><td>■ every 5 sec.</td></tr></table> 	signal duration:	■ every 5 sec.
signal duration:	■ every 5 sec.			

2 Select the Free mode by pressing the Set button. The LED mode indicator at the controller blinks yellow.

i If the LED mode indicator blinks green instead, the signal function is deactivated (see paragraph 11).

Pressing the Key	Display after Pressing the Key	Light Signal
		<p>colour: yellow</p> <p>signal duration: every 5 sec.</p> 

3 Select the Lock mode by pressing the right arrow button. The LED mode indicator at the controller does not blink.

Pressing the Key	Display after Pressing the Key	Light Signal
		<p>no signal</p> 

11. Signal Function for Training Purposes

The signal function for training purposes acoustically supports the patient. When the patient exercises walking with the orthosis, the orthosis provides a tone signal if the signal function is on. This signal indicates if the system joint is locked or free moving.

Via the settings, you can select the tone, the volume and the signal (see paragraph 12.7). You can switch off the signal tone by setting the volume to 0 (see paragraph 12.7.2).

12. Main Menu

In the main menu of the multi-purpose device/app you can set various adjustments for the orthosis. In order to go to the main menu, press the Set button of the multi-purpose device for three seconds. The app leads you via the menu to the same functions.

The following menu items are displayed on the multi-purpose device:

 Pairing (putting the controller into operation)	 Language (not in the app)
 Activate (only for pairing a remote control without Bluetooth)	 Device Info
 Battery Health	 Select Other Leg (only when pairing two controllers with the multi-purpose device)
 Cable Connection Test	 Controller Update
 Settings	 Remote Control Update
 Step Counter	 Activate App

There are two options to leave the menu items of the multi-purpose device:

- 1 If a menu item has more options, exit it via the  back arrow.
- 2 If a menu item has no other options, cancel the selection with **BACK** by pressing the Set button.

12.1 Language

This menu item does not exist in the app as the language is automatically selected by the smartphone's/tablet's settings.

For the multi-purpose device, you can switch the  **Language** between German and English, with German as default language.

In order to change the language of the multi-purpose device, proceed as follows:

- 1 Select the menu item **Language**.
- 2 Select  **German** or  **English**.
- 3 Confirm your selection by pressing the Set button.

When you have confirmed your selection, you will automatically get back to the main menu.

12.2 Pairing (Putting the Controller Into Operation)

In order to establish a connection between the controller and the app, use the app's menu and select the required menu item for a connection with one or two controller(s). Follow the further instructions from the app.

In order to establish a connection between the controller and the multi-purpose device, proceed as follows:

- 1 Select the menu item  **Pairing**.

- 2 Put the controller into operation. With a controller without Bluetooth, you must remove and reinsert a battery. Then, the LEDs on the controller light up briefly. With a controller with Bluetooth, press and hold the MODE switch until a short beep is emitted and, after about 6–10 seconds, a second, longer beep is emitted. Select the **SEARCH** button (fig. 7) if there is not yet a connection between a device and the multi-purpose device.
- 3 **CONFIRM** the found controller within **30 seconds** (fig. 8).
- 4 Select your knee joint system (fig. 9) and press **CONFIRM**.

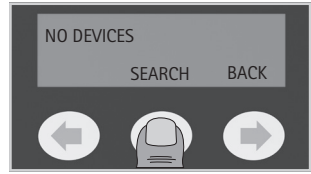
fig. 7

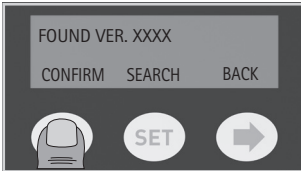
fig. 8

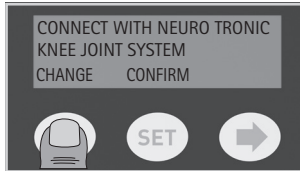
fig. 9

fig. 10

The connection has been successfully established when you can see the message **COMPLETED** on the multi-purpose device display (fig. 10).

Now, you can connect a second controller with the multi-purpose device, if required (fig. 11). If you connect a second controller, legs must be assigned to the controllers. Select the leg for the second controller (fig. 12). Then, the other leg is automatically assigned to the first controller. Repeat points 2 to 4 in order to connect the second controller.

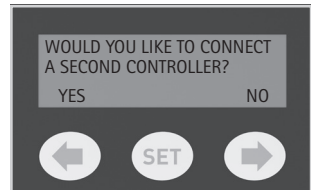
fig. 11

fig. 12

fig. 13

If you have connected two controllers, you can use the menu item **Select Other Leg** to switch between the left and right leg. The currently selected leg is displayed (fig. 13).

12.3 Controller Update

When updating the multi-purpose device/app, the controller update is downloaded simultaneously, if there is an update (see paragraph 15). For the multi-purpose device/app, you can update the chosen controller by selecting the menu item  **Controller Update**. If there is no update, this menu item cannot be selected.

Always keep all controllers in use up to date.

12.4 Remote Control Update

This menu item can only be used if you have a remote control with Bluetooth. When updating the multi-purpose device/app, the remote control update is downloaded simultaneously, if there is an update (see paragraph 15).

In order to check for a remote control update and to perform one, proceed as follows:

- 1 Press the **Auto**, **Free** and **Lock** button of the remote control at the same time for about four seconds (fig. 14).
- 2 Select the menu item **Remote Control Update**.
- 3 If an update is available, confirm the remote control update.

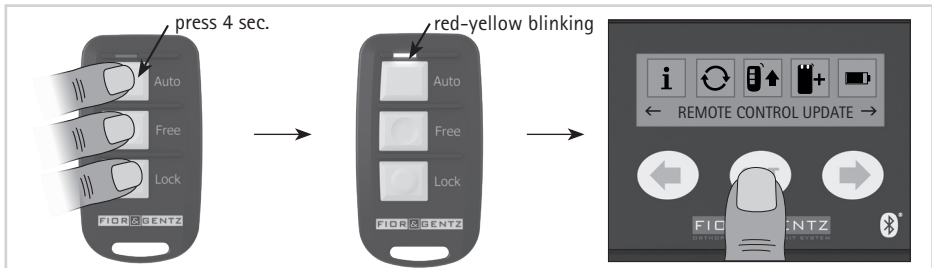
fig. 14

Updating the remote control has been successful when you can see the message **COMPLETED** on the multi-purpose device display. The LED of the remote control blinks green once.

Always keep all remote controls in use up to date.

12.5 Activate (Putting the Remote Control Into Operation)

This menu item is only available for a connection between the multi-purpose device and a remote control **without** Bluetooth. Via the menu item **Activate** you put the remote control for the patient into operation. You establish a connection between the controller and the remote control with the multi-purpose device.

A remote control with Bluetooth can be directly connected with the controller and does not have to be activated with the multi-purpose device (see **Instructions for Use for Orthotists or Qualified/Trained Experts System Knee Joints**).

12.5.1 For One Orthosis

To activate the remote control, proceed as follows:

- 1 Select the menu item **Activate**.
- 2 Press the Auto and Lock button of the remote control at the same time for about four seconds (fig. 15).
- 3 When the LED of the remote control blinks red, the buttons were not pressed at the same time. The remote control cannot be activated. Try again (fig. 15).

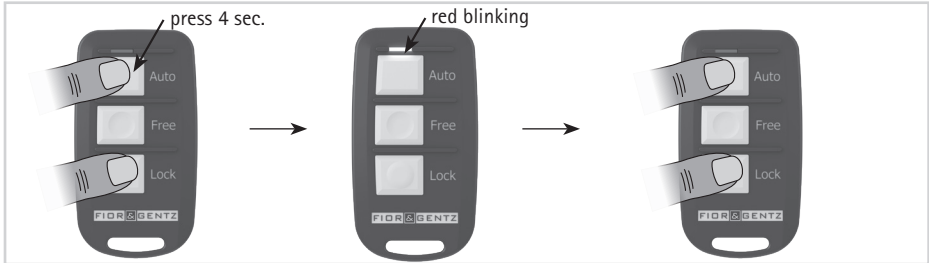
fig. 15

- 4 When the LED blinks yellow, the remote control can be activated.
- 5 Press the Set button of the multi-purpose device in order to activate the remote control (fig. 16).

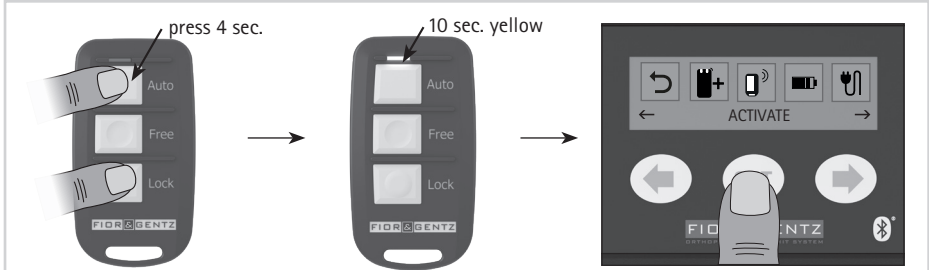
fig. 16

Activating the remote control has been successful when you can see the message **COMPLETED** on the multi-purpose device display. The LED of the remote control blinks green once.

Display	Light Signal Remote Control	Meaning
	colour: green	The remote control has been activated.
	signal duration: ■■■	

12.5.2 For Two Orthoses

When treating both legs, you can decide to connect the controllers of both orthoses with one or two remote control(s). If you activate two remote controls, you can change the modes separately for each controller/orthosis. If you activate only one remote control, the modes of both controllers/orthoses are changed simultaneously.

If the orthoses are equipped with controllers with Bluetooth, the modes can be switched separately or at the same time for the two controllers/orthoses with the **Expert** app as well as the **User** app.

Activating Two Remote Controls for Two Controllers

After you have activated one remote control for one controller (see paragraph 12.5.1) you can activate a second remote control.

Proceed as follows:

- 1 Select the menu item  **Select Other Leg**.
- 2 Select the menu item **Activate**.
- 3 Repeat points 2 to 5 for the activation of one remote control for one orthosis (see paragraph 12.5.1) in order to activate the second remote control.

Activating One Remote Control for Two Controllers

After you have activated one remote control for one controller (see paragraph 12.5.1) you can, within 60 seconds, activate this remote control for a second controller as well.

Proceed as follows:

- 1 Select the menu item **Select Other Leg**.
- 2 Select the menu item **Activate**.
- 3 Press the Auto and Lock button of the remote control at the same time for about four seconds.
- 4 When the LED of the remote control does not blink or blinks red or yellow only once, pressing the buttons at the same time has failed. The remote control cannot be activated. Try again or repeat points 2 to 5 for the activation of one remote control for one orthosis (see paragraph 12.5.1).
- 5 When the LED blinks yellow, the remote control can be activated.
- 6 Press the Set button of the multi-purpose device in order to activate the remote control for both controllers.

If the remote control for the second controller is not activated within 60 seconds after the activation for the first controller, the connection with the first controller is deleted and the second controller is connected instead. In this case, repeat the points of paragraph 12.5.1 and 12.5.2. Mind the given time.

12.6 Cable Connection Test

With this test, you can check the cable connection to the solenoid on the orthosis. For this test, put the orthosis on a workbench. Select the menu item  Cable Connection Test. Then, you receive the result of the cable connection test for the solenoid.

When you start the cable connection test, the orthosis switches automatically into Lock mode and remains in this mode after the test. To change the mode, use the remote control/app or the multi-purpose device/app.

12.6.1 Cable Connection Test Result Messages and Further Actions

The following result messages refer to system components that are explained more detailed in the **Instructions for Use for Orthotists or Qualified/Trained Experts System Knee Joints**. These messages are displayed in the multi-purpose device as well as in the app.

Result Message	Meaning	Further Action
connection of one solenoid OK	The cable connection between the controller and the solenoid is correct.	-
connection of two solenoids OK	The cable connections from both controllers to both solenoids are correct.	-
solenoid not connected	The cable connection between the controller and the solenoid is not correct.	Check the cable connection between the controller and the solenoid.
solenoid short circuited	There is a short circuit in the cable from the controller to the solenoid.	Check the cable connections on the controller and on the solenoid.
C91: Call Technical Support.	Internal device error	Contact Technical Support.

12.7 Settings

In the menu item  Settings you can adjust the following settings on the orthosis:

Calibrate

Rotation Safety

Volume

Select Other Leg
(only with two connected controllers)

Signal Selection

Unlocking in Mid Stance

Tone Selection

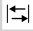
Locking in Terminal Swing

12.7.1 Calibrate

In order for the motion sensors in the controller to detect the position of the lower leg, calibrate the orthosis for a first functional test before fitting. Then, repeat the calibration process. Have the patient wear the orthosis when you calibrate again.

Proceed as follows:

- 1 Select the menu item **Settings**.
 - 2 Select the menu item  **Calibrate**.
 - 3 Place the orthosis with shoe vertically on an even surface (fig. 17). Hold the orthosis. Pay attention to the basic alignment of the orthosis (see paragraph 5.1).
 - 4 Select **CONTINUE** (fig. 18).
 - 5 Place the orthosis horizontally on an even surface (fig. 19). Hold the orthosis. Respect the individual external rotation of the foot.
 - 6 Select **CONTINUE** (fig. 20).
 - 7 If the calibration procedure is finished, select **EXIT**.
- You then leave the menu item **Calibrate** (fig. 21).

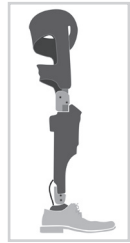
fig. 17

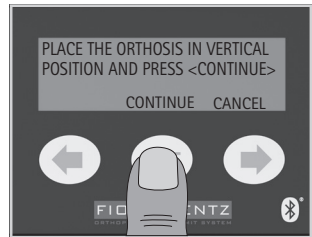
fig. 18

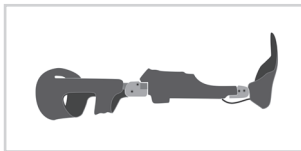
fig. 19

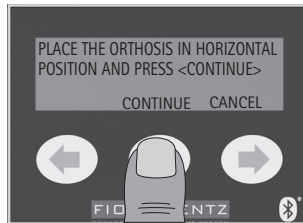
fig. 20

fig. 21

12.7.2 Volume

Here you can set the volume of the signal tone for the signal function for training purposes for the patient (see paragraph 11).

In order to adapt the volume, proceed as follows:

- 1 Select the menu item **Settings**.
- 2 Select the menu item  **Volume**.
Select the volume level.
- 3 For no tone, select volume 0.
For the loudest tone, select volume 5.
- 4 **CONFIRM** your selection (fig. 22).

fig. 22

12.7.3 Tone Selection

Here you can select the tone for the signal function for training purposes for the patient (see paragraph 11). You can select one out of two frequencies. This way, you can select a different tone for each controller/orthosis when both patient's legs are treated.

In order to select a frequency, proceed as follows:

- 1 Select the menu item **Settings**.
- 2 Select the menu item  **Tone Selection**.
- 3 Select the tone.
For a low tone, select frequency 1.
For a high tone, select frequency 2.
- 4 **CONFIRM** your selection (fig. 23).

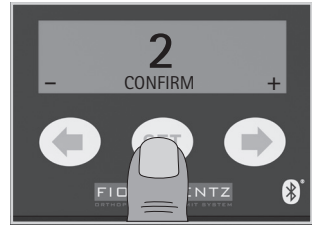
fig. 23

12.7.4 Signal Selection

Here, you can select the signal type for the signal function for training purposes for the patient (see paragraph 11). Signal type 1 is preset.

In order to determine when the signal tone is emitted, proceed as follows:

- 1 Select the menu item **Settings**.
- 2 Select the menu item  **Signal Selection**.
- 3 Select the type of signal. You can select one out of four different setting options.

Signal tone 1 emits a signal as long as the system joint is free moving.

Signal tone 2 emits a signal as long as the system joint is locked.

Signal tone 3 emits a short signal when the system joint switches from free moving to locked and when it switches from locked to free moving.

Signal tone 4 emits two short signals when the system joint switches from free moving to locked and when it switches from locked to free moving.

- 4 **CONFIRM** your selection (fig. 24).

fig. 24

12.7.5 Unlocking in Mid Stance

With this menu item, you can fine adjust the point of time when the system knee joint unlocks in mid stance (see *Instructions for Use for Orthotists or Qualified/Trained Experts System Knee Joints*).

Proceed as follows:

- 1 Select the menu item **Settings**.
- 2 Select the menu item  **Unlocking in Mid Stance**.
- 3 Change the point of time when the system joint unlocks by pressing the arrow buttons.
Press the minus sign in order to unlock the system joint earlier. Press the plus sign in order to unlock the system joint later.
- 4 **CONFIRM** your selection (fig. 25).

fig. 25

Have the patient exercise walking with the orthosis with the adjusted point of time of unlocking and adapt the settings again, if required.

12.7.6 Locking in Terminal Swing

With this menu item, you can fine adjust the point of time when the system knee joint locks in terminal swing (see *Instructions for Use for Orthotists or Qualified/Trained Experts System Knee Joints*).

Proceed as follows:

- 1 Select the menu item **Settings**.
- 2 Select the menu item  **Locking in Terminal Swing**.
- 3 Change the point of time when the system joint locks by pressing the arrow buttons.
Press the minus sign in order to lock the system joint earlier.
Press the plus sign in order to lock the system joint later.
- 4 **CONFIRM** your selection (fig. 25).

Have the patient exercise walking with the orthosis with the adjusted point of time of locking and adapt the settings again, if required.

12.7.7 Rotation Safety

Here, you have the option to adjust the orthosis in a way that enables the system knee joint to unlock easier during rotation movements. The rotation safety is preset to 0 and applies to patients with physiological rotation. Patients with unphysiological rotation movements of the leg inwards or outwards may experience problems with unlocking, which is why you can set an easier unlocking in this menu item.

Proceed as follows:

- 1 Select the menu item **Settings**.
- 2 Select the menu item  **Rotation Safety**.
- 3 Press the plus sign in order to set an easier unlocking. Press the minus sign in order to move back towards normal unlocking for patients with physiological rotation.
- 4 **CONFIRM** your selection (fig. 26).

fig. 26

12.7.8 Select Other Leg

When both legs are treated, you can select the other leg here.

Proceed as follows:

- 1 Select the menu item **Settings**.
- 2 Select the menu item  **Select Other Leg**.
- 3 Select the leg.
- 4 **CONFIRM** your selection (fig. 27).

fig. 27

12.8 Battery Health

Here, you can check the health of the battery/batteries in the controller with the multi-purpose device/app. The patient must not wear the orthosis when the battery health test is performed.

With a controller with integrated lithium-polymer battery, the battery health is immediately apparent on the display/in the app. If the battery health is no longer sufficient, exchange the controller. Do not try to disassemble the controller, as the battery is an integral part of the controller.

With a controller with exchangeable batteries, the batteries have to be fully charged for the test. The battery health test cannot be carried out with two solenoids at the same time. If you have connected two system knee joints to one controller, disconnect one solenoid to carry out the test.

In order to check the battery health, proceed as follows:

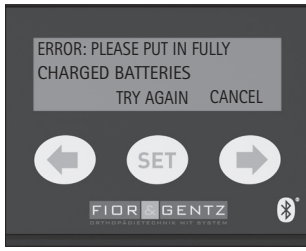
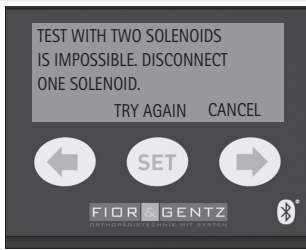
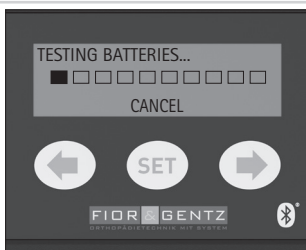
- 1 Insert fully charged batteries into the controller.
- 2 To go to the main menu of the multi-purpose device, press the Set button for three seconds.
- 3 Select the menu item  **Battery Health**.
- 4 Select **CONTINUE**.

If you want to cancel the test, select **CANCEL** on the display.

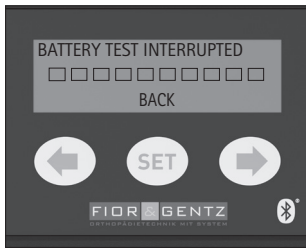
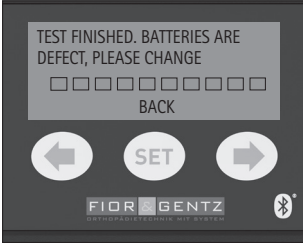
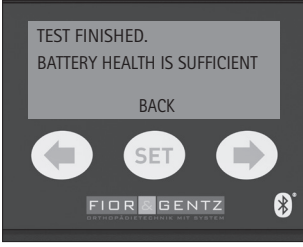
Depending on the battery health, the test can take up to five hours.

The following messages can be displayed when the test is started:

Message (Display)	Meaning	Further Action
	<p>The batteries in the controller are not fully charged.</p>	<p>Please insert fully charged batteries in the controller and select TRY AGAIN.</p>
	<p>You have mounted two system knee joints in the orthosis. The battery health test cannot be carried out with two connected solenoids.</p>	<p>Disconnect one solenoid.</p>
	<p>The battery health is being tested.</p>	<p>Wait for 2 hours and check the display again.</p>

The following messages can be displayed during or after the test:

Message (Display)	Meaning	Further Action
	<p>The test has been interrupted, e.g. because the batteries have become loose from the holder or the battery health is too bad to continue the test.</p>	<p>Exchange the batteries for new ones and make sure that the controller/orthosis is stored in a way that the batteries cannot become loose from the holder. Repeat the test.</p>

Message (Display)	Meaning	Further Action
	The test is finished after 2 hours maximum. The battery health is bad.	Exchange the batteries for new ones.
	The test is finished after 2 hours. The battery quality is sufficient.	You can cancel the test and use the batteries or wait until the test is finished.
	The test is finished after 5 hours. The battery quality is sufficient.	The test has shortened the period of use of the batteries. Charge the batteries completely before using them again.

12.9 Step Counter

The controller saves and displays the steps that your patient has made with the leg with orthosis.

If both legs of the patient are treated and settings are made for one leg, the currently selected menu item and the selected leg are displayed alternately.

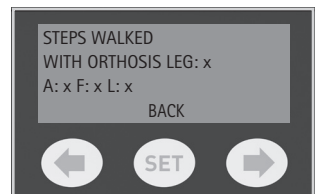
fig. 28

The step counter is available to you as an orthotist or qualified/trained expert via the multi-purpose device as well as the Expert app and to the patient via the User app.

In order to have the steps displayed in the multi-purpose device, select the menu item  Step Counter. The steps made with the orthosis in the different modes (A = Auto mode, F = Free mode, L = Lock mode) will be displayed (fig. 28). For the total number of steps, double the indicated value.

12.10 Device Info

Here, you can find information on the version of the multi-purpose device/app and the controller. In order to have this information displayed in the app, select the corresponding menu item. In order to have this information displayed in the multi-purpose device, select the menu item **i** Device Info. You will see the versions of the currently used multi-purpose device and the controller (fig. 29).

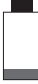
You will also find a code for troubleshooting under this menu item. In case of any problems, you can give this code to FIOR & GENTZ so that we can help you faster. To display the code, go to MORE.

fig. 29

13. Indication of the Battery Status in the Multi-Purpose Device/App

In the multi-purpose device, the controller's battery status is indicated in the device's upper right corner. In the app, you can see the controller's battery status as well. The following symbols concerning the battery status can be displayed:

View		Meaning
		The batteries in the controller are fully charged.
		The batteries in the controller are 1/3 full.
		The batteries in the controller are almost empty.

14. Replacing the Batteries of the Multi-Purpose Device

The multi-purpose device has two batteries of the type CR2450.

In order to replace the batteries, proceed as follows:

- 1 Remove the four grey cover caps on the back of the multi-purpose device. There are screws underneath.
- 2 Unscrew the four screws with a PH1 screwdriver.
- 3 Take off the cover of the multi-purpose device.
- 4 Remove the two batteries.
- 5 Insert two new batteries of the type CR2450 with the positive pole facing up.
- 6 Put the cover on again.
- 7 Screw in the screws.
- 8 Put the grey cover caps on again.

15. Updating the Multi-Purpose Device/the Expert App

If you want to save battery power, you can connect the multi-purpose device via USB cable to a PC or to a mobile phone charger. The multi-purpose device uses no battery power while it is connected.

Depending on the settings of your smartphone/tablet, updates are downloaded and installed automatically or you have to update your apps separately. If you have deactivated automatic app updates for your smartphone/tablet, you should regularly check for updates.

You can find the updates for the multi-purpose device in the download section on our website at www.fior-gentz.com.

In order to update the multi-purpose device, proceed as follows:

- 1 Download the update.
- 2 Open the file.
- 3 Connect the multi-purpose device to the computer with the USB cable.
The multi-purpose device turns on.
- 4 Click on "Start Update" in order to update the multi-purpose device.

You must not turn off the multi-purpose device during the updating process. If the updating process has been successful, the current device version is displayed. Enter each update with the version number on the back page of these instructions for use for your records.

16. Advice on Optimal Orthosis Functionality

16.1 Bluetooth Connection

In a knee joint system with Bluetooth, the quality of connection depends on how interference-free your environment is.

16.2 Walking with the Orthosis in Auto Mode

For safety reasons, the system knee joint remains locked during the first step. When the patient starts to walk from a standing position, they should make the first step with the leg with orthosis. The electronics require a stride to switch from standing mode to walking mode in order to avoid disturbed gait or stumbling when starting to walk.

16.3 Remote Control

Problem	Further Action
The controller does not respond to pressed buttons on the remote control.	Check whether the controller is still connected to the Expert or User app and whether the patient is standing still with the orthosis. If the problem remains, contact Technical Support.
The LEDs on the controller do not blink when pressing a button on the remote control.	

16.4 Multi-Purpose Device

Problem	Further Action
The display is blank.	Contact Technical Support.
The device does not always react when pressing a button.	
The display is soiled.	Clean the display with a dry cloth.

16.5 Controller

Problem	Further Action
Only applies to a controller with exchangeable batteries: The LEDs do not blink when batteries are inserted.	Check the polarity and health of the batteries. If the problem remains, contact Technical Support.
Only applies to a controller with an integrated lithium-polymer battery: The LEDs do not blink after pressing the MODE switch.	Charge the battery completely. If the problem remains, contact Technical Support.
No devices are found during the pairing of the controller.	Connect the multi-purpose device/app with the controller within 30 seconds . Check whether the LEDs light up or whether a short and a longer beep tone can be heard (see paragraph 12.2). If the problem remains, contact Technical Support.

17. Disposal

Dispose of the product and its individual parts properly. The product must not be disposed of with the residual waste (fig. 30). Please comply with the applicable national laws and local regulations for the proper recycling of recyclable materials.

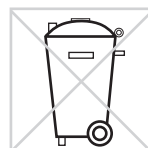
fig. 30

18. CE Conformity

We declare that our medical devices as well as our accessories for medical devices are in conformity with the requirements of Regulation (EU) 2017/745. Therefore, the FIOR & GENTZ products bear the CE marking.

19. Legal Information

With the purchase of this product, our General Terms and Conditions of Business Transactions, Sales, Delivery and Payment will apply. The warranty expires, for example, if the product is mounted several times. Please note that the product is not supposed to be combined with other components or materials than with those recommended by the FIOR & GENTZ Orthosis Configurator. Should the product be combined with products from other manufacturers, a written consent by the seller is required.

The information in these instructions for use is valid at the date of printing. The contained product information serves as a guideline. Subject to technical modifications.

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